



## Follow the steps below to ensure a successful test:

### 1. LOG IN & ACTIVATE YOUR KIT

To get started please create your account on **basepaws.com** and then activate your kit. Your activation code is on the tape.



### 2. COLLECT DNA

- Carefully peel the tape from its backing
- Place tape on your cat (the tag facing tail)
- Pet your cat nicely to make sure you get more hair
- Gently remove the tape (pull back)
- Repeat all steps with the second tape
- Place adhesive in return envelope facing each other, hair side in



### 3. MAIL IN

Mail the sample to us in the return envelope. **Please make sure you activated your kit before sending it.** We will notify you when we have received it.



**Note:** We extract DNA from the root of your cat's hair. Make sure to get as much hair root on the adhesive as possible by pulling **against the root** (it will not hurt the cat!).